



## Cleaning Procedures

### Upholstery – Spot Cleaning & Extraction

1. Identify stain and use the stain chart on [www.cryptoncare.com](http://www.cryptoncare.com) to select the appropriate cleaner, Crypton Care Gold (Protein cleaner) or Crypton Care Purple (Ink, Grease and Iodine cleaner).
2. Remove any excess residue with the back of the Crypton upholstery brush. Spray the spot, lightly brush with the soft bristled Crypton upholstery brush, allow the cleaner to sit for a minute, and blot with a clean, dry towel.
3. If spot still remains, repeat the spray, brush & blot process.
4. For stubborn stains, thoroughly wet the surface with the cleaners (both), brush and blot.
5. To keep upholstery looking new, use the Crypton Upholstery Extraction concentrate at a rate of 4 oz of concentrate per gallon of water in Crypton Care “Ruby” extractor or a similar water extraction machine to periodically extract the upholstery.
6. Always rinse well with clean water from the extractor to remove any residue.

### Disinfecting Crypton:

To kill bacteria, remove odors and prevent mold and mildew, the newly EPA approved Crypton Fabric Disinfectant and Deodorizer should be used. Extract using a water extraction machine, like the Crypton Care “Ruby” extractor, to remove the soiled spot. Apply a mist of Crypton Fabric Disinfectant and Deodorizer to the surface and keep the area wet for ten (10) minutes. Crypton Fabric Disinfectant and Deodorizer is a quaternary disinfectant, virucidal, and will prevent mold and mildew when used as directed. This process will provide a Crypton surface that is germ free!

**Note:** This hospital grade disinfectant can be used on other hard, inanimate surfaces for the same disinfecting qualities.